Super Dandy "Inhalierboy"
Art.-Nr. 156800

WWW.WALDHAUSEN.COM

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Easy handling and efficient treatment – safe and easy to handle

Effective medicament dispenser.

Plug the connecting hose to the compressor's hose connection with a slight turn. Connect the other, smaller end of the hose to the nebuliser.

Connect the compressor to the power supply system. Start by connecting the AC power line to the compressor's plug socket. Next plug the power plug to a suitable power socket.

Unscrew the upper of the nebuliser and fill in the required amount of the medicine. The maximum filling capacity is 8 ml, minimum filling capacity: 2 ml. Srew upper tightly back to the medicine bowl.

Please do not use essential oils or alcoholic dilutions!

Turn the nebuliser on by pushing the compressor's on/off switch. Please take care that the ventilation slots at the compressor's sides remain unimpeded.

Please take a sensitive approach when using the inhalator for the first time or when applying to sensitive horses. We recommend that you acquaint your horse to the inhalator before the first application: push the nebuliser's contact breaker and let your horse inhale the emissive vapour.

When using the inhalating basket, you need to connect it to the hose connection on top of the nebuliser. The nebulising hose may also be used as a kind of air bag nebuliser. Simply hold the hose closely to the horse's left or right nostril.

Cleaning

Please make sure the compressor is turned off and please disconnect machine from power supply before cleaning! The compressor's outside may be cleaned with a damp towel. Do not apply water or other liquids to the ventilation slots!

Cleaning of the nebuliser: disassemble nebuliser and clean the separate parts with warm water and washing-up liquid. Please rinse thoroughly! For disinfection: boil for 15 minutes in a sufficient amount of water. Dry on a lintless cloth. The nebuliser is dishwasher safe, may be concoced, and disinfected. Please replace all parts of the nebuliser by no later than one year, as the material may undergo changes in the course of time. Please take note that contamination causes germs.

Filter change

The Super Dandy Inhalator's front panel filter lasts up to 200 operating hours under normal regular working conditions, but should nevertheless be replaced at least once a year. Check the filter at regular intervals for dirt, blockage or dampness. In case the filter is filthy, damp or blocked it needs to be replaced. Unsrew the filter (use a coin and turn counterclockwise) and insert a new filter.

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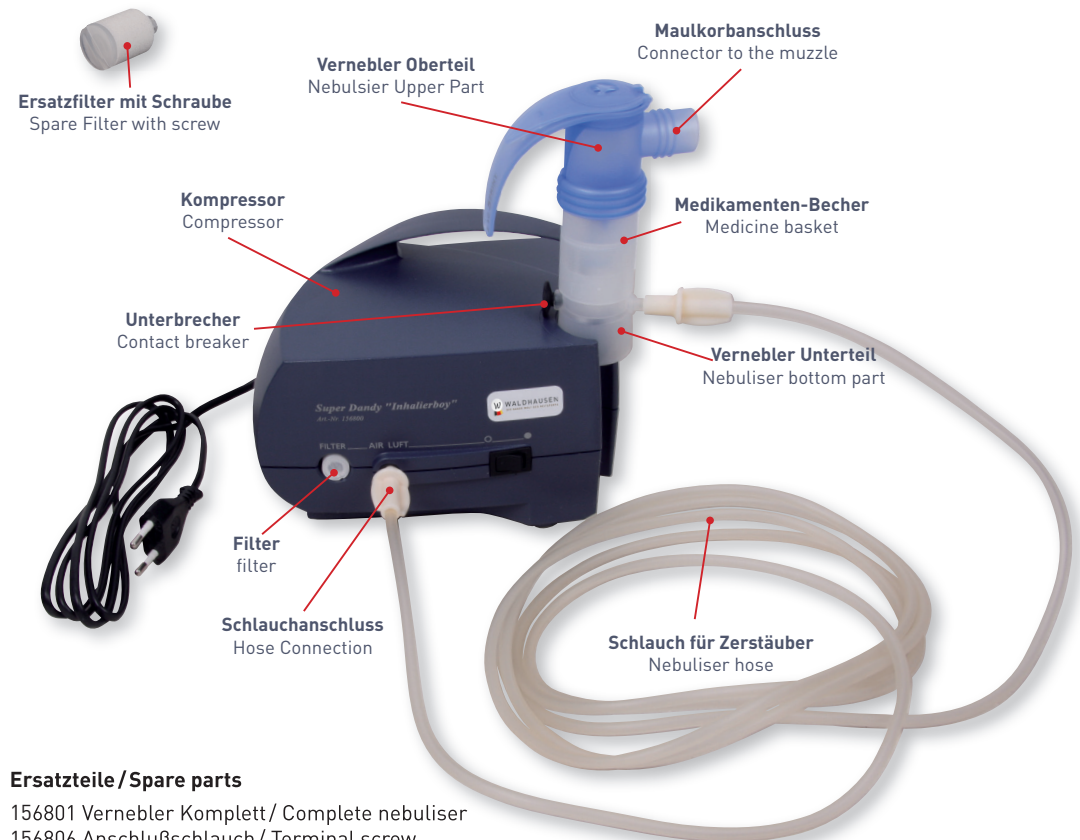
Maintenance of the connecting tube

Turn on the compressor and keep the machine going until all humidity inside the hose has been removed.

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